## MESCOMP TECHNOLOGIES

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# ENGLISH

## **XXIV. Features:**

- Big LCD display.
- User's blood pressure settings.
- Irregular heartbeat detector
- 3 users x 30 memories
- Storing last 30 measurements with date and time for each user.
- Range of measurement: pressure 0~300 mmHg
- Pulse: 40~199 beats/minute
- Auto power off 1 minute after last key operation
- Measurement systolic, diastolic, pulse

## **XXV. Important information, please read before use the unit!!!**

- Blood pressure measurements should be interpreted by a physician or trained health professional who is familiar with your medical history. By using the unit regularly and recording the results for your physician to interpret, you keep your physician informed of the continuing trends in your blood pressure.
- Wind the cuff snugly around your wrist and the cuff must be at the same level as your heart.
- Do not vibrate the unit during measurement, or the proper measurement will not be achieved.
- Perform measurement quietly in a relaxed position.

- Do not wind the cuff over jacket or sweater sleeve, or measurement cannot be done.
- Keep in mind, that blood pressure naturally varies from time to time through out the day and also is affected by lots of different factors such as smoking, alcohol consumption, medical mentation and physical activity.
- People with a condition that causes circulatory problems (diabetes, kidney disease, arteriosclerosis or poor peripheral circulation) may get lower readings with this monitor that with a blood pressure monitor that is used on the upper arm. Please consult your physician to determine if your wrist blood pressure accurately reflects your actual blood pressure.
- Blood pressure measurement determine with the unit are equivalent to those obtained by a trained observer using the cuff / stethoscope auscultation method. Within the limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.
- Use a piece of cloth with water or mild cleansing agent to clean the case and then use a piece of dry cloth to wipe it dry.
- Do not use gas or any strong cleansers in cleaning.
- When the unit is not to be used for a long time, remove the batteries. (Leaking of battery liquid can cause trouble)
- Always keep the unit in the carrying case after its use.
- Do not put the item directly under the sunlight, in high temperature, or humid and dusty places.

- Do not store in extremely low (less than - 20 °C) or high (more than 70 °C) temperature.
- Do not in any way twist the wrist cuff.
- Do not start the air pressure when the wrist cuff is not wrapped around the wrist.
- Do not attempt to disassemble or change any parts of the monitor including wrist cuff.
- Do not drop the product or put it through strong impact.
- The device measures the blood pressure of adult only.
- The device is not supposed to be used if your wrist has any wound or injury.
- In the event of the cuff pumping up and not stopping, please open the cuff at once.

## XXVI. Specification:

Model number	: AC 229
Method of measurement	: Oscillometric
Range of Measurement	: Pressure 0 ~ 300 mmHg pulse 40 ~ 199 beats/minute
Accuracy calibration	: Pressure +/- 3 mmHg, Pulse +/- 5%
Fuzzy Pressure	: Automatic Re-inflation pump system
Inflation	: Automatic Inflation ( Air pump)
Deflation of Pressure	: Automatic air release control valve
Exhaust	: Automatic exhaust vale
Sets of memory	: 3 databases of 30 memory each, total 90 sets
Unit weight	: approx 133g
Operating temperature	: +10 °C ~ +40 °C, less than 85% R.H.
Storage temperature	: -20 °C ~ +70 °C, less than 85% R.H.
Power supply	: 2 x "AAA" (1.5V) alkaline battery
Accessories	: Carrying case, instruction manual 2 pcs "AAA"(1.5V) battery

## **XXVII. Important information's about blood pressure.**

### **What is blood pressure?**

Blood pressure is a measurement of the force of blood flowing against the walls of the arteries. Arterial blood pressure is constantly changing during the course of the cardiac cycle. The highest pressure in the cycle is called the systolic blood pressure, the lowest is the diastolic blood pressure. Both pressure readings, the systolic and the diastolic are necessary to enable a physician to evaluate the status of a patient's blood pressure. Many factors such as physical activity, anxiety or the time of day, can influence your blood pressure. Blood pressure is typically low in the morning and increases from the afternoon to the evening. It is lower in the summer and higher in the winter.


### **Why is it useful to measure blood pressure at home?**

Having one's blood pressure measured by a doctor in a hospital or a clinic, and a group health checks, tend to stimulate nervousness in the subject and may even create high blood pressure. Also varies blood pressure in accordance with a variety of conditions and so judgment is not possible on the basis of a single measurement. The blood pressure measured first thing in the morning after getting up, before taking any food and with the subject still, is known as the fundamental blood pressure. In practice it is rather difficult to record the fundamental blood pressure, but to come as near as possible to measuring the blood

pressure in an environment that is close to this, is why it is useful to take the measurement at home.

### WHO blood pressure classifications

Standards for assessment of high or low blood pressure without regard to age, have been established by the World Health Organization (WHO), as shown in the chart.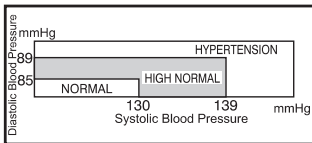


### Variations in blood pressure

Individual blood pressures vary greatly both on a daily and a seasonal basis. These variations are even more pronounced in hypertension patients. Normally the blood pressure rises while at work and is at its lowest during sleeping period. (hypertension: means a person who has high blood pressure symptom.) The graph below illustrated the variations in blood pressure over a whole day with measurement taken every five minutes.


The thick line represents sleep. The rise in blood pressure at 4 PM (A in the graph) and 12 PM (B in the graph) correspond to an attack of pain.

The thick line represents sleep. The rise in blood pressure at 4 PM (A in the graph) and 12 PM (B in the graph) correspond to an attack of pain.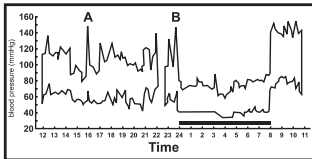


## XXVIII. Device description.

Name of parts

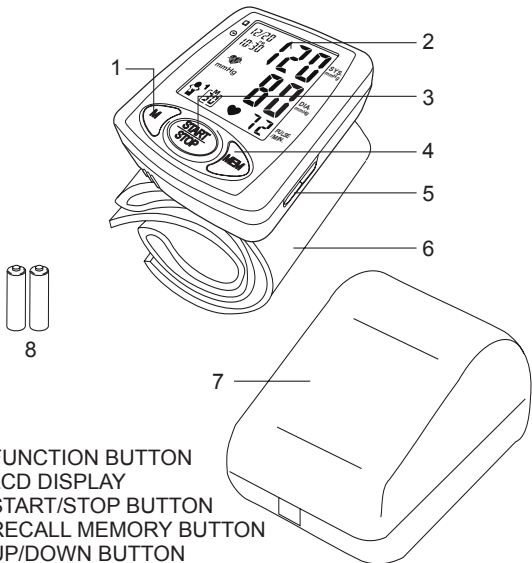- 1.FUNCTION BUTTON
- 2.LCD DISPLAY
- 3.START/STOP BUTTON
- 4.RECALL MEMORY BUTTON
- 5.UP/DOWN BUTTON
- 6.WRIST CUFF
- 7.CARRYING CASE
- 8.ALKALINE BATTERY

## XXIX. Descriptions of display symbols:
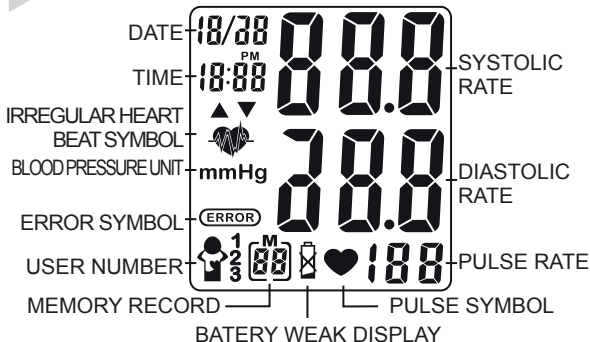
## XXX. Description of display marks

Mark appears in the measurement condition and flashes when pulse is detected.

Appears when the battery voltage is excessively low or the positions of batteries are incorrect.




Appears when the accurate blood pressure could not be obtained accurately.

Appears for 1 minute when the user was talking, moving or shaking or an irregular heart beat was detected during measurement.

## XXXI. Changing batteries

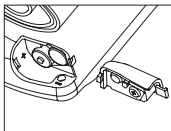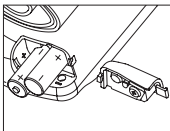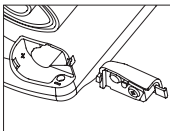- a. Remove the battery cover at the rear side of unit and insert ALKALINE batteries into the battery compartment as shown, taking extreme care that the polarities + and - are observed
- b. All stored memories (if any ) will still be kept after changing batteries.

### To change batteries:

-  When  show up on the display, the battery is weak.
- Change the batteries or it will not be possible to take blood pressure measurement.
- All batteries used must be the same type. Do not mix alkaline, standard(carbon-zinc) or rechargeable (cadmium) batteries.
- Do not mix old and new batteries.

## XXXII. Attaching pressure cuff

### A. Wrap the pressure cuff around the wrist.

- the display of the unit should be placed on the palm side of the wrist.
- the wrist should be bare.

### B. Fasten the pressure cuff snugly.

- do not pull strongly on the pressure cuff.
- do not make the pressure cuff too tight.

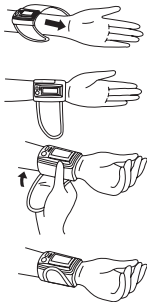
Figure. No. 1

## XXXIII. Correct measurement position

It is extremely important for the measured values, that the cuff is at the same height of the heart. Higher or deeper position will give deviations from the true value.

- Place your elbow on a table or other object (such as our carry case).
- Use the armrest to position the arm so that the pressure cuff is at the same height as the heart.

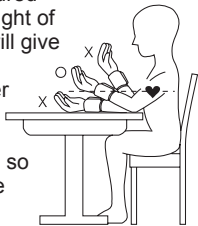
Figure. No. 2

- Relax your hand with the palm facing up.

## **XXXIV. How to measure**

### **A. Inserting batteries in position:**

- Open battery cover.
- Refer to the picture in positioning batteries.
- All LCD segments appear on the display in 3 seconds.
- Display 1month 1 date, 12 hour 0 minute.

### **B. Setting Date and Time:**

- Press " M " button ("month" starts flashing) Press "▲" or "▼" button to set the correct month (1,2,3,.....12)
- Press " M " button again ("date" starts flashing) Press "▲" or "▼" button to set the correct date.
- Press " M " button again ("hour" starts flashing) Press "▲" or "▼" button to set the correct hour in 12-hour format.
- Press " M " button again ("minute" starts flashing) Press "▲" or "▼" button to set the correct minute (0,1,2,3,.....59)
- Press " M " button again 3 times to switch to normal time (month, date, hour and minute appear on the display)

### **C. Setting the SYS and DIA of your bpm – color check feature**

The factory setting for your BPM is for USER 1, as follow:  
SYSTOLIC (SYS) : 130 mmHg  
DIASTOLIC (DIA) : 90 mmHg

For the SYS, it means that when taking your blood pressure, if your SYS is above 130 mmHg, upper part of the screen will turn RED; if your SYS is below 130mmHg, then the upper part of the screen will turn BLUE;

For the DIA, it means that when taking your blood pressure, if your DIA is above 90 mmHg, lower part of the screen will turn RED; if your DIA is below 90 mmHg, then the lower part of the screen will turn BLUE;

If you want to modify this setting, we recommend you to check with your doctor to define your own settings based on your age, general health condition, etc....

To modify the settings of USER 1:

- a. Follow all the steps on paragraph B above (Setting date and Hour) TILL POINT d. If you already set the time and date, simply press 5 times in a row the M (MODE) key.
- b. The SYSTOLIC figure is now flashing on the upper part of screen (130 = factory setting) ; Press the selection key "▲" or "▼" on the side of the unit to adjust up or down this figure (please note that the adjustment is done by +/-5 mmHg each time you press the "▲" or "▼" key).
- c. Press the "M (MODE)" key to confirm setting and go to the setting of the DIASTOLIC.
- d. The DIASTOLIC figure is now flashing on the lower part of screen (90 = factory setting) ; Press the selection key "▲" or "▼" on the side of the unit to adjust up or down this figure (Please note that the adjustment is done by +/-5 mmHg each time you press the "▲" or "▼" key).
- e. Press the "M (MODE)" key to confirm and complete the setting of your Blood pressure monitor. The screen now displays the time and date and the current user number.

**IMPORTANT ! FIRST SETTING OF THE SYS and DIA FOR USER 2 and 3 of your Blood pressure monitor.**

You Blood pressure monitor has multi user feature (up to 3 user); it means that different person can use it and save their records in separate memory, which allow an easier follow up of their records. See the next section 'Multiple users' for more details.

For user 2 and user 3, no settings have been done yet;. WITHOUT doing these settings, you will not see ANY LIGHTING ON THE SCREEN when using your blood pressure monitor under user 2 or user 3. You need to do the setting FIRST if you want to activate the LIGHTING FEATURE for these users.

To do the setting of the SYSTOLIC and DIASTOLIC for user 2 and/or 3:

- a. Select user 2 or 3 (see next paragraph D on how to select user 1, 2 or 3)
- b. Follow all the steps described on paragraph B above (Setting date and Hour) TILL POINT d. If you already set the time and date, simply press 5 times in a row the "M (MODE)" key.
- c. Press the "M (MODE)" Key to display the SYS and DIA figures; The SYS figure on the upper part of the screen is flashing (135).
- d. Press the selection key "▲" or "▼" on the side of the unit to adjust up or down this figure (Please note that the adjustment is done by +/-5 mmHg each time you press the "▲" or "▼" key).
- e. Press the "M (MODE)" key to confirm setting and go to the setting of the DIASTOLIC.

- f. The DIASTOLIC figure is now the flashing on the lower part of the screen (85) ; Press the selection key "▲" or "▼" on the side of the unit to adjust up or down this figure (Please note that the adjustment is done by +/-5 mmHg each time you press the "▲" or "▼" key).
- g. Press the "M (MODE)" key to confirm and complete the setting of your Blood pressure monitor. The screen now display the time and date and the current user number.

#### **D. Multiple User Mode**

Your blood pressure monitor can keep the records of up to 3 different users in separate memory. This is very useful if several members in your family are using your blood pressure monitor. For instance, you may keep all your records in the memory of USER 1, and the all the records of your wife in the USER 2. You may also set different settings for the SYS and DIA as explained in previous paragraph C.

When you first start the unit, default user is USER 1; to select another user, simply press the "▲" or "▼" on the side of the unit to select user 2 or 3. Then, Press the "START/STOP" key to start the measurement of your blood pressure.

#### **E. Steps to take your Blood pressure measurement**

- a. Wrap the cuff around the wrist (refer to wrapping wrist cuff as Figure No. 1)
- b. Sit upright on the chair too have correct posture (refer to correct posture in Taking Blood Pressure as Figure No. 2)
- c. Press the "START/STOP" key to switch on the unit;  
Select the User 1, 2 or 3 as described in above paragraph D.

- d. Then, Press again "START/STOP" key to START the measurement. All the segment of the screen will appear and then the air pressure will automatically pump up to 195 mmHg. If your blood pressure tends to be on the higher level, the air pressure will automatically increase to the required level for measurement. DO NOT MOVE OR TALK DURING THE MEASUREMENT.
- e. After the air pressure has increased, it will slowly decrease. When the pulse rate is detected, the HEART rate symbol will start flashing; the measurement is processing.
- f. At the end of the measure, the SYS, DIA, Heart pulse rate will be displayed for one minute. You may see the Irregular heart beat symbol (see related section Irregular heart beat for more information)

## **XXXV. Memory – Special feature ‘Average of the last 3 measures’**

After each measurement, the SYS, DIA and heart pulse rate will be automatically stored in memory together with the time and date. Each Memory can keep records up to the latest 30 measures. If more than 30 measures, the memory will automatically clear out the earliest data.

### **a. AVERAGE of the LAST 3 MEASURES**

Once you first press the "MEM" key, the average of the last 3 measurements is displayed; the symbol [A] (Average) is shown to remind you that this is the average of the last 3 measures.

This feature is interesting as, for example, by taking your blood pressure every morning in the same condition, you can evaluate more precisely your usual blood pressure level.

b. MEMORY

Press the "MEM" key to recall the past measurements in memory. The first information displayed is the Average of the past 3 measurements (see above); Press again "MEM" key to see the last measurement results in memory, and so on. Each measurement in memory is identified with a number from 1 to 30, the time and the date.

Press the "START/STOP" key to stop reading the measurements in memory.

Please note that the Irregular heart beat symbol is NOT stored into memory

c. ERASING RECORDS in MEMORY

(a) Press the "▲" or "▼" key to select user 1, 2 or 3

(b) Press the MEM key

(c) HOLD the "M (MODE)" Key and the "▲" simultaneously to erase all information in Memory for the selected user. You should see the letter 'CLA' on the screen during the process.

## XXXVI. Irregular heart beat feature

We call 'Arrhythmia' the irregularity of cardiac rhythm; it means that your heart beat is irregular, either too fast or too slow. The appearance of symbol, signifies that a certain pulse irregularity was detected during measurement.

Talking, moving, shaking or irregular pulses can result the appearance of this symbol. Arrhythmia can affect the result of your blood pressure measurements.

Usually this is not a cause for concern; however, if the symbol appears almost, we recommend you to seek medical advice.

**The device does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.**

**The irregular heart beat detector symbol appears under below condition:**

If user is talking, moving, shaking or having irregular pulses during the measurement,  symbol will appear like Figure No. 3. However, the BPM has been able to display results.

Figure No. 3

If user is talking, moving, shaking or having irregular pulse during the measurement, and when the accurate blood pressure could not be obtained, both  and **ERROR** symbol will appear like Figure No. 4.

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